Topester

Topester Band
Nastro Topester
Páska Topester

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EN **Instructions For Use** (04)

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Topester

Topester Band

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Topester Band

Description

Topester Band is a non-absorbable, flat-woven and uncoated tape, chemically consisting of polyethylene terephthalate. The tape is available in white or green (dyed), in a variety of widths and lengths. Topester Band is not attached to a needle.

Application

Topester Band can be used as a retraction and/or fixing tape during surgery, or as a temporary fixation aid for vessels or tissue and for temporary ligation of vessels. The flat structure of the tape minimizes the risks of tissue damage or cuts.

Combination With Other Medical Devices

Topester Band may be used in combination with tubing segments and surgical thread snares (Tourniquet-Set) or surgical clips.

Contraindications

None known.

Warnings

Retention of the product in the body must not exceed 24 hours. Topester Band is a single use device and may not be reused due to hygienic hazards. Reuse of Tourniquet-Sets can cause infections or cross-infections.

Precautions

Contaminated or infected wounds must be treated following acceptable surgical practice.

Adverse Reactions

Like all foreign bodies used as surgical material, Topester Band may aggravate an existing infection.

How Supplied

Topester Band is packaged in sterile wrapping and is available in a variety of widths and lengths.

Symbols

Use until (YYYY-MM)

See Instructions for Use

Sterilized by Ethylene oxide

Batch Number

Article Number

Manufacturer

Do not re-sterilize

Do not reuse

Store dry

Do not expose to sunlight

Do not use if package is damaged

CE-mark and identification number of
Notified Body

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Santec Medicalprodukte GmbH